

Instructions Manual

Goid


WKS034.1.A26.DB / WKS034.1.A26.LB

Electric Spa Bed

VERSION 3.25

1.Welcome!

Congratulations on purchasing this new spa bed. This manual is only for reference of using and operating this electric spa bed. Our company does not assume all the consequences and responsibilities caused by using this manual for other purposes.

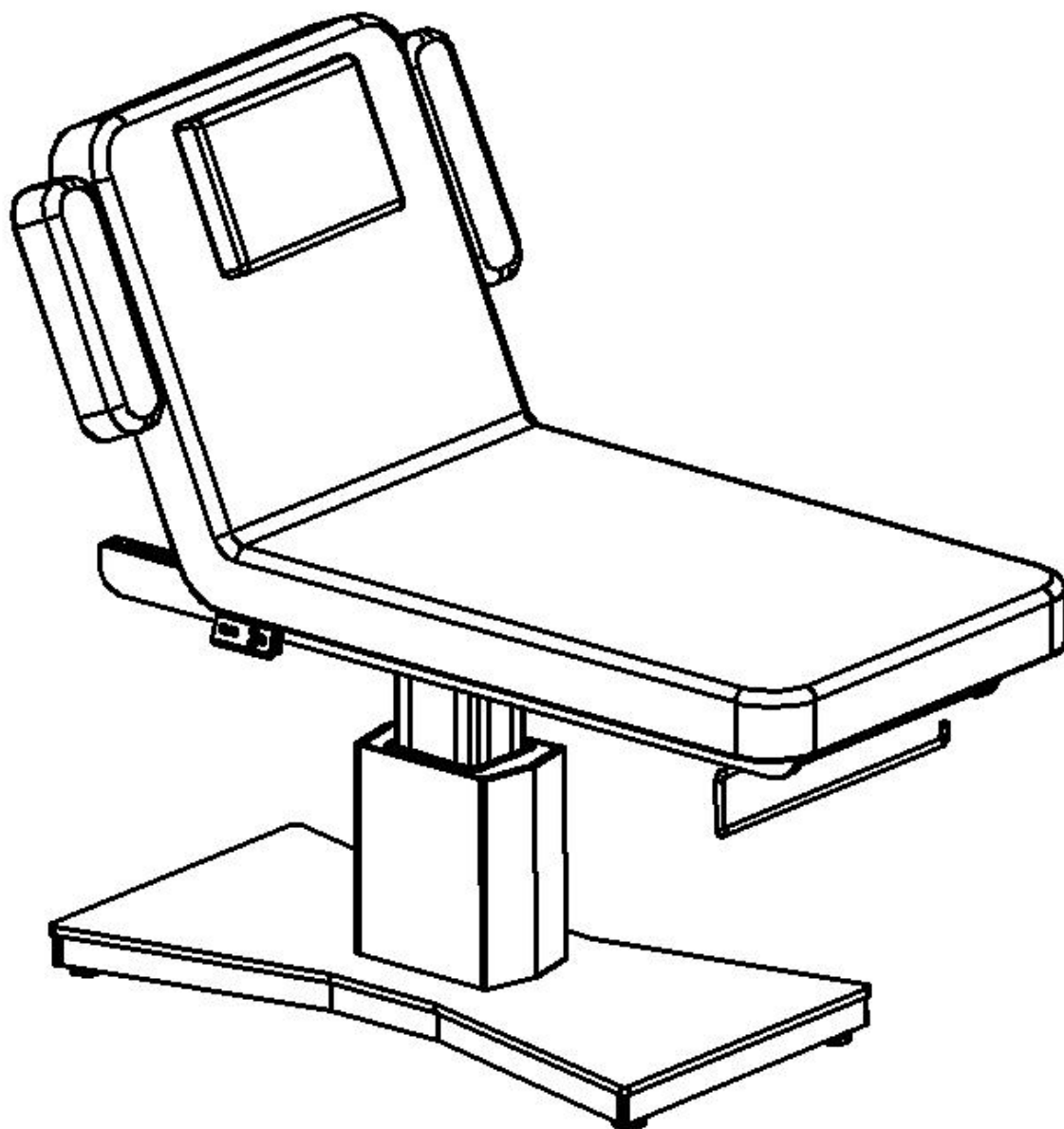

WK[®]


Imported by
WEELKO BARCELONA SL

Pol. Ind. La Vall d'an
Camí de Garreta, 92
08600 Berga
Barcelona - Spain
T. +34 93 824 90 20
E. info@weelko.com

2.Warning

 Protective grounding sign

 Functional ground identification

 This electric beauty bed is classified as Class II BF type equipment according to the protection against electric shock hazards.

- ☑ Power supply must be cut off when changing electrical components or maintaining and cleaning machines.
- ☑ The plug must be plugged into an outlet that is properly installed and grounded, to prevent the risk of fire and electric shock.
- ☑ Do not stand on this equipment.
- ☑ Do not allow children to play with and place the equipment in a place where children can avoid touching; Please adjust the backrest back to the highest position when you leave.
- ☑ Please do not place the equipment in damp places such as bathroom to avoid electric shock or other failure; The remote control should also avoid contact with water.
- ☑ Please do not place the equipment in places where painting/coating or oxygen providing during operation.
- ☑ Unless being monitored by guardians or instructed and responsible for their personal safety, the people who have physical obstacles, tactile disorders, mental disorders, lacking of experience, common sense or children, are not allow to use the equipment.

- ☑ Please contact distributor, or related manufacturer with appropriate skill and experience for repairing if the plug is damaged, in case of any dangerous.
- ☑ Please make sure there is no accessible thing within the stroke during any operation of the beds.
- ☑ Maintenance should be processed by trained professionals.

3.Precautions

Read this manual carefully before installation, operation and adjustment, and please keep it well.

This electric beauty bed is equipped with a three-wire power cord. The power cord must be plugged into an three-hole outlet which is properly installed and grounded. This is not only to prevent the risk of electric shock, but also ensure bed work properly.

While controlling the lifting of the whole bed, the button must be pressed continuously until the bed be in a suitable position.

Please pay attention that the input frequency should be $M_s \times 6\text{min/h}$ when operating the motor.

Maintenance should be processed by trained professionals.


Please use neutral reagent to clear the dust, stains on leather or on plastic cover after each treatment.

4. Structure of electric spa bed

This beauty bed consists of one motor controller box, motor, hardware, plastic parts and PU leather. Among them, the controller box is controlling the operation of the motor and hand controller.

This beauty bed includes angle adjustment of the backrest and lifting adjustment of whole bed which is an overall and integral structure.

(Details as shown below)

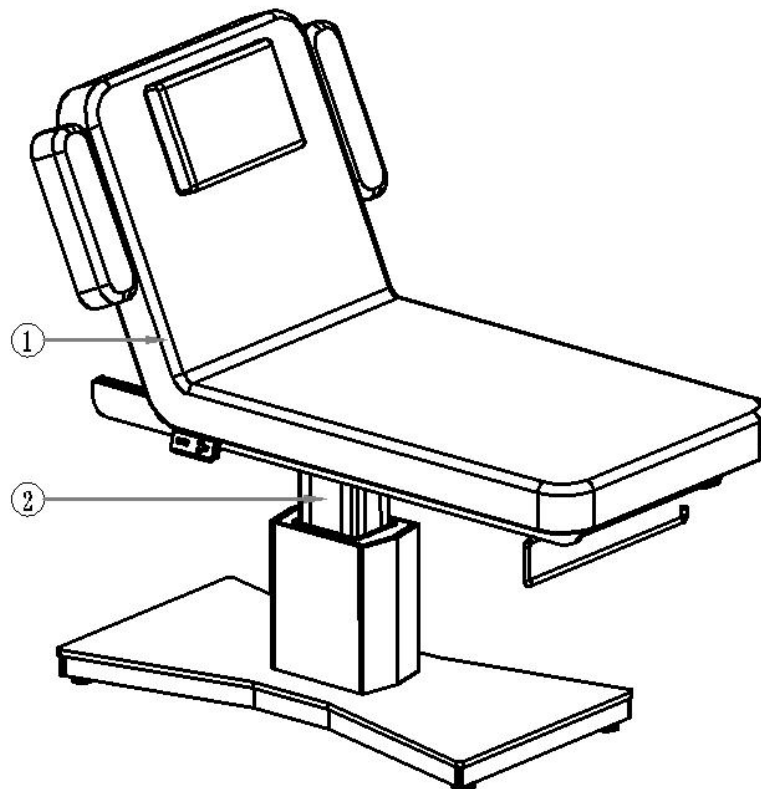


- ① Angle adjustment of the backrest
- ② Adjustment of the whole bed height

This electric beauty bed is suitable for beauty and massage treatment.


5.Installation

- ① Put the bed to a proper location, according to the overall layout, the Day lighting or artificial lighting and operation condition of the salon's or home. And keep it in the clean, dry and cool operating environment.
- ② Open the packing carton to check if the bed is intact and see if the accessories or spares are complete. Kindly contact distributor or manufacturer if any problem.
- ③ Place the bed to the proper position. Please note that it is unnecessary to install extra screw on the floor to fasten the bed, because the original design has take the stability performance into consideration. But please ensure the bed is placed on a firm and flat ground, so as to prevent any accident. If the bed rotates or slant due to rough ground, you could adjust the four pieces of plastic feet (M10×30) and nuts (M10) by hexagon open-end wrench, which are fasten on the bottom panel, so as to ensure the bed is stable.
- ④Power Input: Voltage and frequency input shall be AC 100-120V/60Hz, 220-240V/ 50Hz. Plug the bed on the power supply properly.
(Details as shown below)

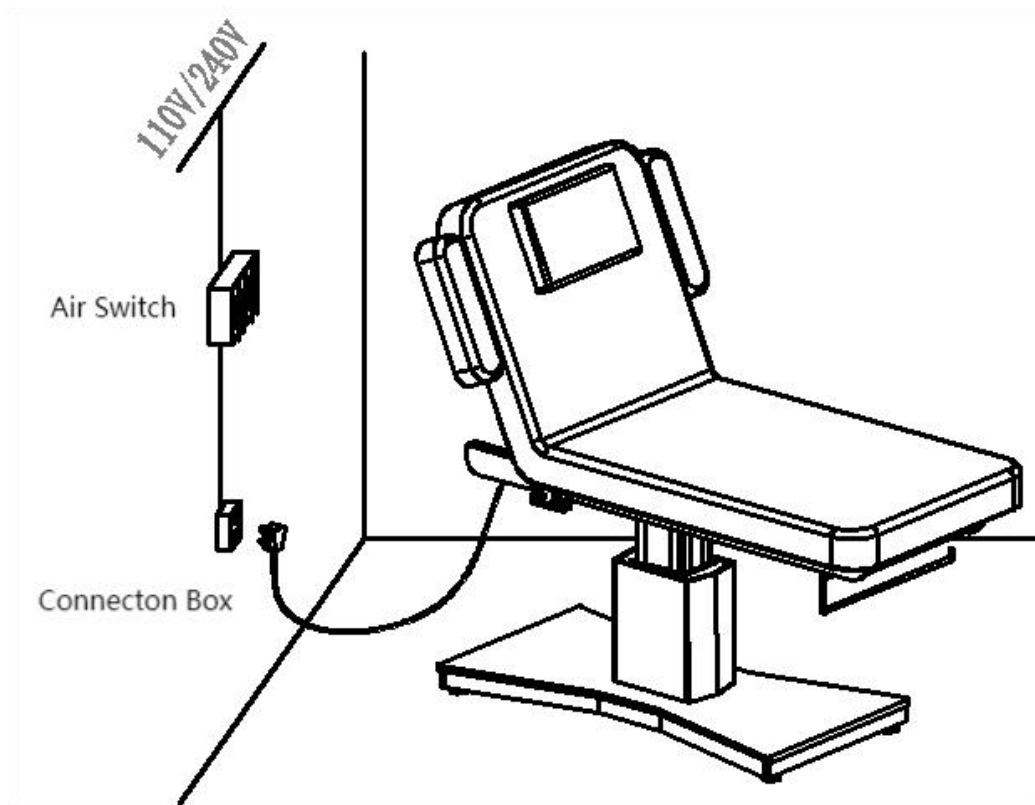

6.Operation instruction


Standard configuration (Instruction of Hand Control)

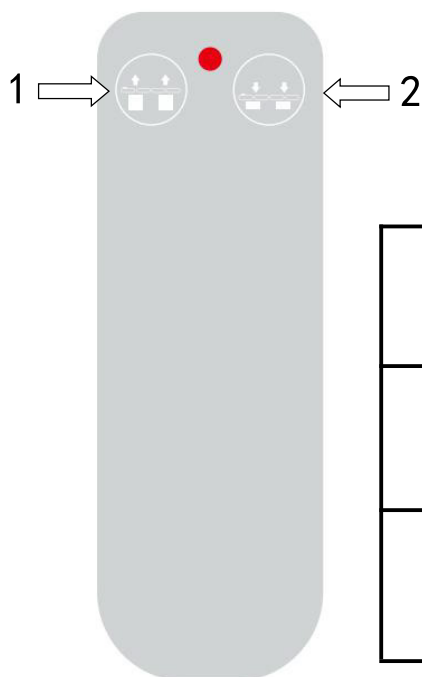

| | Description | Operation |
|---|---------------------------|--|
| 1 | Whole bed motor goes up | The whole bed goes up smoothly by pressing the button. |
| 2 | Whole bed motor goes down | The whole bed goes down smoothly by pressing the button. |

7. Technical Parameters

| | |
|----------------------------|---|
| Product size | 1980(L)×1010(W)×600(H)mm |
| Package size | 2020×820×630mm |
| The highest Position | 850±10mm |
| The Lowest Position | 600±10mm |
| Net weight | 75kg (±2KG) |
| Gross weight | 88kg (±2KG) |
| Power | AC 100~120V ,50Hz/60Hz or AC 220~240V, 50Hz/60Hz to chice |
| Backrest adjustment | 0°~+67° (±10%) |
| Footrest adjustment | - 25°~ 0° (±10%) |
| Handrest adjustment height | 130mm |
| Lifting rated load | 200 kg |

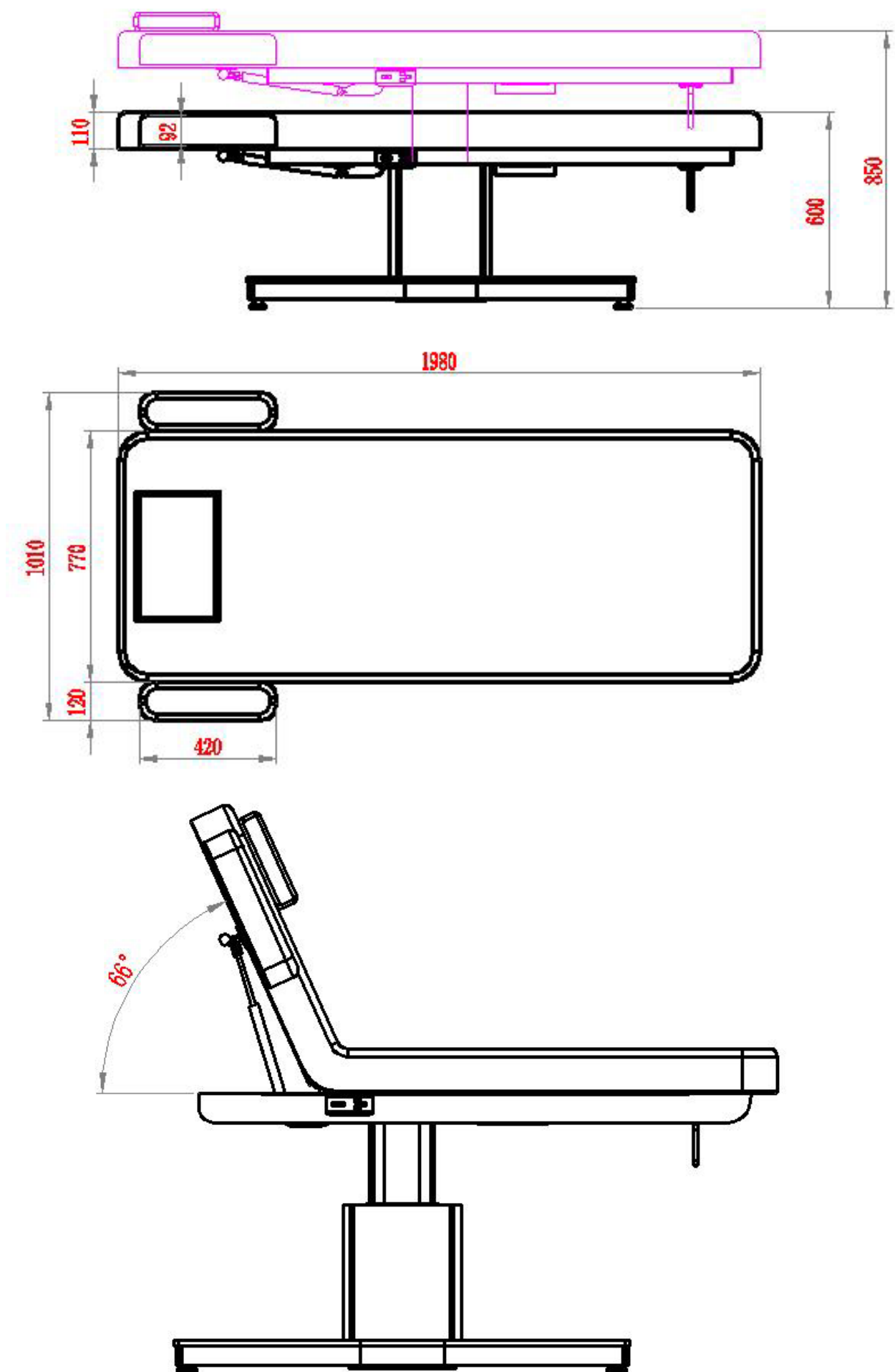


8.Maintenance

After every treatment, please use neutral reagent to clear the dust, stains on leather or the plastic cover. This is to prevent premature aging of the leather and plastic cover.

9.Circuit system connection



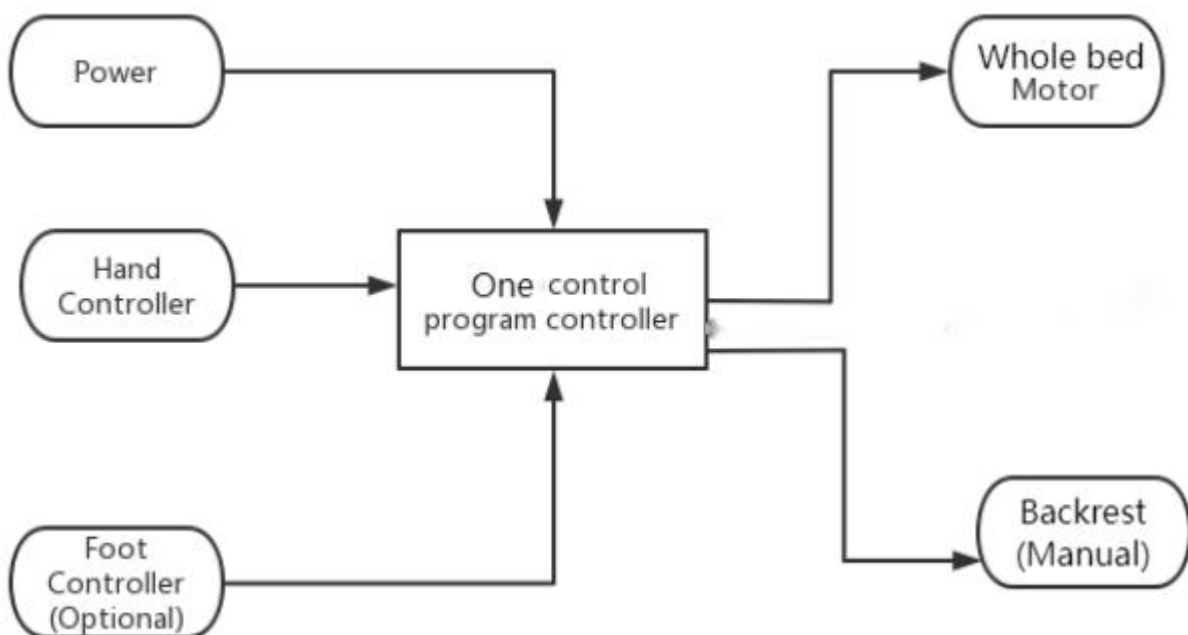


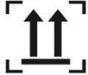




AC 100-120V ,50Hz/60Hz or AC 220-240V ,50Hz/60Hz



10.Troubleshooting

| Problems | Reason | Solution |
|------------------------------|--|---|
| Can't go up or down normally | The power cord is unconnected or not well plugged | Connect the power cord or plug it well |
| | Poor quality of manual controller | Replace the manual control panel |
| | Poor quality of foot controller switch | If problem is still not solved after replacing manual or foot control switch, kindly contact maintenance center or supplier to clear the trouble. |
| | Poor quality of controller box | |
| | Rupture of motor tail | |
| | Motor detonation | |
| Abnormal sound | The rotating parts lack of lubricating oil | Add lubricating oil |
| | Rotating position (such as hinge, P, O, M sets) Lack of lubricating oil | Add lubricating oil |
| | Motor quality problems | Replace the motor |

11. Icon explanation

| | | | |
|---|--|---|---|
|  | Warning note, refer to accompanying documents |  | Manufacturer |
|  | Protective grounding |  | Maximum safe load |
|  | Type B application part of the equipment |  | Upward |
|  | Protection from rain |  | Separate disposal sign for waste electrical and electronic equipment (please comply with local laws and regulations) |
|  | Battery |  | Place carefully and gently |

Kindly check and verify products specifications before use and this manual is only used for reference. The product specifications will be subject to change or update without further notice.

